


# Range of Motion Hinged Knee Brace Instructions


## STEP 1

Range of Motion hinge is set at 0° at the factory. If the Range of Motion setting needs to be adjusted, remove hinge from pocket while taking care not to drop the small locking pin from the center of the hinge. Move the small pin to the hole showing the desired setting.

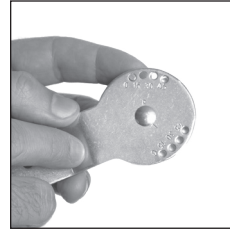


## STEP 2

For Open Wrap Hinged Knee Brace, Model #8\*278, position brace behind knee, pull wraps around to front of leg and secure. The knee patella opening should sit comfortably over the patella (knee cap). Hinge pockets should be positioned adjacent to the knee on either side.

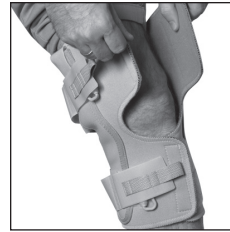

**Model #8\*278 ROM  
Open Wrap Hinged Knee Brace**

For Hinged Knee Brace, Model #8\*275, pull on brace so the knee patella opening is sitting comfortably over the patella (knee cap). Hinge pockets should be positioned adjacent to the knee on either side.

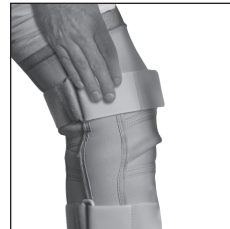

**Model #8\*275 ROM  
Hinged Knee Brace**

## STEP 3

Insert Range of Motion hinge back into hinge pocket while taking care not to drop the small locking pin from the center of the hinge. Repeat for both sides.



## STEP 4

Once hinges are positioned in brace, secure brace with support straps to ensure a snug fit.



**Swede-O is proud to be the U.S. distributor of Thermoskin products.  
Thermoskin® & Trioxon® are registered trademarks of  
United Pacific Industries Pty LTD.**

**Swede-O Inc.**  
6459 Ash Street  
North Branch, MN 55056  
Toll Free: (800) 525-9339  
Phone: (651) 674-8301 | Fax: (651) 674-8425  
Web: [www.swedeo.com](http://www.swedeo.com)